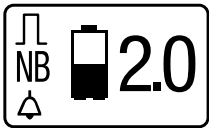
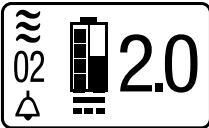
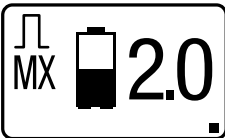
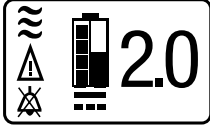

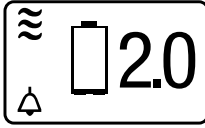
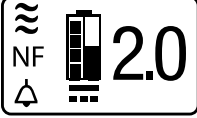
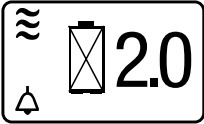





Alarm Indicators and Screen Symbols




Visual, Audio Indicators	Description	What To Do
 <p data-bbox="440 831 574 953">3 short beeps + 1 long beep</p>	<p data-bbox="651 562 849 590">No Breath Alarm</p> <p data-bbox="651 600 963 1041">This alarm occurs when a breath is not detected for a period of 2 minutes or more. It can be silenced by pressing the Alarm symbol. This alarm becomes silent as soon as a breath is detected. If no breath is detected after approximately 30 minutes, the device shuts down to conserve power. (This alarm is inactive for ten minutes after startup.)</p>	<p data-bbox="1005 562 1214 1041">Check the connection from the cannula to the device. Make sure the nasal cannula is properly positioned on your face and that you are breathing through your nose. Make sure the cannula tubing is not kinked or obstructed.</p>
 <p data-bbox="440 1268 574 1360">3 long beeps + 1 short beep</p>	<p data-bbox="651 1094 899 1150">Low Oxygen Concentration Alarm</p> <p data-bbox="651 1161 963 1350">This alarm occurs when the device is delivering a lower concentration of oxygen than specified. This alarm can be silenced by pressing the Alarm symbol.</p>	<p data-bbox="1005 1144 1214 1297">Change to another source of oxygen and contact your equipment provider.</p>
 <p data-bbox="423 1644 591 1703">1 long beep after 2 minutes</p>	<p data-bbox="651 1402 922 1430">High Breath Rate Alarm</p> <p data-bbox="651 1440 963 1713">This alarm indicates that the user's breath rate is exceeding the capacity of the device. The device is still working properly and is still providing oxygen at the rate specified for the setting. The beeps can be silenced by pressing the Alarm symbol.</p>	<p data-bbox="1005 1434 1214 1686">The indicator resets itself when the breath rate is reduced. If this indicator regularly occurs, contact your home care provider.</p>





SimplyGo User Manual

Visual, Audio Indicators	Description	What To Do
 <p data-bbox="430 762 581 856">3 short beeps + 3 long beeps</p>	<p data-bbox="651 499 906 527">Technical Fault Alarm</p> <p data-bbox="651 541 959 730">This alarm occurs when: a) the battery runs out or b) the device has a general malfunction and the device is no longer operating properly.</p> <p data-bbox="651 741 954 835">The device may or may not shut down depending on the severity of the alarm.</p> <p data-bbox="651 846 943 940">This alarm can be silenced if the  symbol appears on-screen.</p>	<p data-bbox="1008 562 1214 657">Recharge the battery and try to restart the device.</p> <p data-bbox="1008 667 1219 888">If this alarm occurs again, change to another source of oxygen and contact your home care provider.</p>
 <p data-bbox="435 1203 581 1234">1 short beep</p>	<p data-bbox="651 982 873 1010">Low Battery Alarm</p> <p data-bbox="651 1024 943 1276">This alarm occurs when approximately 17 minutes of battery life remains. Remaining battery life is dependent on your device settings and your activity level. The empty battery symbol flashes on-screen.</p>	<p data-bbox="1008 1035 1219 1224">Replace the battery or connect to a power source. (Press the Alarm symbol to silence the alarm.)</p>
 <p data-bbox="435 1486 581 1581">3 long beeps + 1 short beep</p>	<p data-bbox="651 1360 829 1388">No Flow Alarm</p> <p data-bbox="651 1402 943 1528">This alarm occurs when the device detects there is no oxygen flowing in the patient cannula.</p>	<p data-bbox="1008 1350 1219 1539">Check the cannula for kinks or other obstructions that are stopping oxygen flow through it.</p>


Visual, Audio Indicators	Description	What To Do
 <p>Alternating short and long beeps</p>	<p>Depleted Battery Alarm</p> <p>This alarm occurs when approximately two minutes of battery life remains. The battery symbol flashes on-screen.</p>	<p>Replace or recharge battery. (Press Alarm symbol to silence the alarm.)</p>
	<p>Alarm Silence Symbol</p> <p>This symbol appears when you press the Alarm Silence button to silence the audible alarm (beeps).</p>	<p>No further action is needed.</p>
	<p>Attention</p> <p>This symbol appears when corrective action is required.</p>	<p>Allow the device to warm up. If the alarm continues, contact your home care provider.</p>
	<p>Battery Charging While Almost Empty Symbol</p> <p>This battery symbol appears (with the charging bars scrolling on the left side) when the battery is almost depleted and recharging.</p>	<p>No action needed.</p>

SimplyGo User Manual

Visual, Audio Indicators	Description	What To Do
	<p>Battery Charging While Partly Full Symbol</p> <p>When a battery symbol similar to this appears (with the charging bars scrolling on the left and at mid-height on the right), the battery is partly full and recharging.</p> <p>The left side of the symbol indicates charging, and the right side indicates the battery level.</p>	<p>No action needed.</p>
 1 long beep	<p>Battery Charge Complete Symbol</p> <p>This battery symbol appears when the battery is fully charged.</p>	<p>No action needed.</p>
	<p>Power Connection Symbol</p> <p>This symbol appears when the device is connected to an AC, DC, or airline power source.</p>	<p>No action needed.</p>
<p>(Power Connection Symbol Disappears)</p> <p>1 short beep</p>	<p>The Power Connection symbol disappears when the device is disconnected from the power source.</p>	<p>No action needed.</p>

Visual, Audio Indicators	Description	What To Do
 <p>1st press of the Power button = 1 long beep.</p> <p>2nd press of the Power button to turn device OFF = 3 long beeps</p>	<p>On/Off Symbol</p> <p>This symbol appears when the device is powered on, or is ready to be turned on or off.</p>	<p>Press the Power button to turn the device on or off.</p> <p>Note: To prevent inadvertent starts and stops, the device requires two presses of the Power button to turn it on and off.</p>
	<p>Pulse Symbol</p> <p>The dot appears in the lower, right hand corner of the display screen when the device delivers a pulse, whenever you take a breath.</p>	<p>No action needed.</p>
	<p>Pulse Mode Indicator</p> <p>This symbol appears when the device is operating in Pulse mode.</p>	<p>If this is your Mode setting, no action is needed.</p> <p>If this is not your Mode setting, press the Mode button until your proper Mode setting appears.</p>
	<p>Sleep Mode Indicator</p> <p>This symbol appears when the device is operating in Sleep mode.</p>	<p>If this is your Mode setting, no action is needed.</p> <p>If this is not your Mode setting, press the Mode button until your proper Mode setting appears.</p>

SimplyGo User Manual

Visual, Audio Indicators	Description	What To Do
	<p>Continuous Flow Mode Indicator</p> <p>This symbol appears when the device is operating in Continuous Flow Mode.</p>	<p>If this is your Mode setting, no action is needed.</p> <p>If this is not your Mode setting, press the Mode button until your proper Mode setting appears.</p>
<p>2.0</p> <p>1 short beep</p>	<p>Flow Control Setting</p> <p>The large number on the right side of the screen shows the oxygen flow setting. (In this example, it is 2.) This number can range from 1 to 6 in increments of one-half for Pulse and Sleep modes and from .5 to 2 for Continuous Flow mode.</p>	<p>If this is your flow setting, no action is needed.</p> <p>If this is not your flow setting, press the + button to increase this number or press the – button to decrease this number.</p>

Troubleshooting

The table below lists common problems and actions you can take. If you are unable to resolve a problem, please contact your home care provider.

Problem	Possible Cause	What You Should Do
Device Won't Turn On	Battery is depleted.	Use the AC or DC power cords to operate the device (with the battery inserted) to recharge the battery. If this does not resolve the problem, contact your home care provider.
	Battery is not installed correctly.	Remove the battery and re-install it correctly.
	Power button not pressed twice.	Press the power button twice.
Device Will Not Trigger A Pulse Of Oxygen	SimplyGo is not turned on.	Press the power button twice.
	Cannula tubing is kinked or twisted.	Make sure the tubing is connected properly to the oxygen outlet port and that it is free of any obstruction.
	Device malfunction.	Contact your home care provider.
Oxygen Not At Full Concentration	Device is warming up.	Wait 10 minutes for the device to deliver oxygen at the prescribed concentration. If the condition persists, contact your home care provider.
Alarm Occurs	The device needs your attention.	See the Alarm Indicators and Screen Symbols section for information about specific alarms and what you should do.